GC Tooth Mousse®

TOPICAL CREME WITH BIO-AVAILABLE CALCIUM AND PHOSPHATE

For use by or under the supervision of a dental professional in the recommended indications.

Description
GC Tooth Mousse is a water based creme containing RECALDENT™™ (CPP-ACP: Casein Phosphopeptide - Amorphous Calcium Phosphate). When CPP-ACP is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxypapatite and soft tissue localising bio-available calcium and phosphate.

Important:
Saliva will enhance the effectiveness of CPP-ACP and the flavour will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

Topical paste containing bio-available calcium and phosphate

1. Provides extra protection for tooth.
2. Helps neutralize acid challenges from acidogenic bacteria in plaque.
3. Helps neutralize acid challenges from other internal and external acid sources.

CONTRAINDICATIONS
RECALDENT™™ (CPP-ACP) is derived from milk casein. Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzate preservatives.

DIRECTIONS FOR USE

- INDICATIONS**
- a. Prior to and following in-office bleaching.
 - b. After ultrasonic, hand scaling or root planing.
 - c. Following professional tooth cleaning (P.T.C.).
 - d. After application of topical fluoride.
 - e. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome.
 - f. During orthodontic treatment.
 - g. For high risk caries patients.
 - h. For special needs patients.

In-office application:

1. Custom tray application

1. Before use rinse the custom tray thoroughly under running water.
2. Extrude a generous layer of GC Tooth Mousse into the tray and apply to the upper and / or lower teeth.
3. Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
4. Remove the tray.
5. Instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Instruct the patient to retain for as long as possible (1 - 2 minutes) avoiding expectorating and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
6. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

Any residual GC Tooth Mousse in the tray should be rinsed or brushed off under running water immediately after use.

II. Non-tray application

1. If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is **NOT** necessary to dry the teeth with compressed air.
2. Apply a sufficient amount of GC Tooth Mousse to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas using an Interproximal Tooth Cleaning Brush.
3. Leave GC Tooth Mousse on the surface for a minimum of 3 minutes.
4. Then instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1 - 2 minutes) avoiding expectoration and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
5. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

III. Day time application

after tooth brushing as recommended by a dental professional

1. Apply a sufficient amount of GC Tooth Mousse to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.
2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1 - 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

IV. Night time application

after tooth brushing as recommended by a dental professional

1. Apply a sufficient amount of GC Tooth Mousse to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse.
2. Leave GC Tooth Mousse in the mouth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (a further 1 - 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

FLAVORS

Strawberry, Melon, Vanilla, Mint, Tutti-Frutti

STORAGE

For optimal performance, store in a dry location at room temperature (15-25°C/59-77°F).

(Shell life: 2 years from date of manufacture)

PACKAGES

1. Tube containing 40g (35mL)
2. Assorted Packags
10 x 40g (2 each of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)
2. Single Flavor Package
10 x 40g in each flavor Vanilla, Strawberry, Melon**, Tutti-Frutti**, Mint **available in Europe only
3. Refill Package
1 x 40g tube in each flavor

NOTE

GC Tooth Mousse with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne Victoria Australia. *RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada, and USA.

CAUTION

1. Always replace cap after use and remove any residual paste or moisture around the tube neck or inside the cap.
2. In case of contact with eyes, flush immediately with water and seek medical attention!
3. In case of spillage on clothes, wash off with water.
4. If any allergic symptoms are experienced, this may indicate sensitivity or allergy to the benzate preservatives, or to some other component of the product such as a flavoring agent. In this event, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheet available at: <http://www.geeurope.com/>

They can also be obtained from your supplier.

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For use by and under the supervision of a dental professional in the recommended indications.

Description
GC Tooth Mouse is a water based creme containing RECALDENT™ (CPP-ACP; Casein Phosphopeptide – Amorphous Calcium Phosphate). It provides extra fluoride, zinc, calcium, vitamins B1, B6, B12, biotin, beta-carotene, vitamins A, C, E, and zinc. GC Tooth Mouse will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localising bio-available calcium and phosphate.

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A topical paste containing bio-available calcium and phosphate
1. Provides extra protection for teeth.
2. Helps neutralise bacteria from acidogenic bacteria in plaque.
3. Helps neutralise acid challenges from other internal and external acid sources.

CONTRAINDICATIONS
RECALDENT™ (CPP-ACP) is derived from milk casein. Do not use the product on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzoate preservatives.

DIRECTIONS FOR USE
INDICATIONS
Prior to and following in-office bleaching
a. After ultrasonic, hand scaling or root planing
c. Following professional tooth cleaning (P.T.C.)
d. Following orthodontic treatment
e. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome.
f. During orthodontic treatment.
g. For high risk patients with a high susceptibility to erosion.
h. For special needs patients.

In-office application:

1. **Custom tray application**
a. Before using the custom tray thoroughly under running water and apply to the upper and / or lower teeth.
b. Apply the tray undisturbed in the mouth for a minimum of 3 minutes.
c. Remove the tray.
5. Instruct the patient to use the tongue to spread the remaining GC Tooth Mouse throughout the mouth. Instruct the patient to keep the tongue flat against the roof of the mouth for as long as possible (an additional 1 – 2 minutes) avoiding expectoration and delaying swallowing. The longer GC Tooth Mouse and saliva are maintained in the mouth, the more effective the result.
6. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mouse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
7. Any residual GC Tooth Mouse in the tray should be rinsed or brushed off under running water immediately after use.

II. Non tray application:

1. If necessary, remove any excess saliva on the tooth surface with a compressed air. However, it is NOT necessary to dry the teeth with compressed air.
2. Apply a sufficient amount of GC Tooth Mouse to the tooth surfaces using an application brush.
3. Leave GC Tooth Mouse undisturbed for a minimum of 3 minutes.
4. Then instruct the patient to use the tongue to spread the remaining GC Tooth Mouse throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1 – 2 minutes) avoiding expectoration and delaying swallowing. The longer GC Tooth Mouse and saliva are maintained in the mouth, the more effective the result.
5. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mouse remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

At home application:
II. Day time application after tooth brushing as recommended by a dental professional
1. Apply a sufficient amount of GC Tooth Mouse to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mouse.
2. Leave GC Tooth Mouse on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mouse throughout the mouth. Hold in the mouth for as long as possible (a further 1 – 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mouse and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mouse remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

IV. Night time application after tooth brushing as recommended by a dental professional
1. Apply a sufficient amount of GC Tooth Mouse to the upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mouse.
2. Leave GC Tooth Mouse on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mouse throughout the mouth. Hold in the mouth for as long as possible (a further 1 – 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mouse and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mouse remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

At home application:
III. Day time application after tooth brushing as recommended by a dental professional
1. Apply a sufficient amount of GC Tooth Mouse to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mouse.
2. Leave GC Tooth Mouse on the teeth undisturbed for a minimum of 3 minutes.
3. Then use your tongue to spread the remaining GC Tooth Mouse throughout the mouth. Hold in the mouth for as long as possible (a further 1 – 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mouse and saliva are maintained in the mouth, the more effective the result.
4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mouse remaining on the surface can be left to gradually dissipate overnight. Do not eat or drink for 30 minutes following application.

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NOTE:
GC Tooth Mouse with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne Victoria Australia
RECALDENT is a trademark used under license. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada, and USA.

CAUTION
1. Always replace cap after use and remove any residual paste or moisture around the tube neck in inside the cap.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In case of spillage on clothes, wash off with water.
4. If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzoate preservatives, or to some other component of the product or to the preservative. In this event, discontinue the use of the product and refer to a physician

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